Drew Scientific, Park Road, Barrow in Furness, Cumbria, UK

Glycated Haemoglobin Two Level Calibrator Kit

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For in vitro diagnostic use only. These calibrators are intended for use with instruments for the measurement of HbA1c and are designed to allow calibration to either the American NGSP system or the International IFCC system.

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HbA1c measurement is used clinically to assess glycaemic control in diabetes. Calibrators should be used regularly to ensure that HbA1c measurements are aligned with acknowledged local and international reference systems. The values assigned to these calibrators have been carefully determined with reference to NGSP and IFCC reference materials. These materials should be used in accordance with the calibration recommendation made by the supplier of the users HbA1c test method.

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Contents, Contenu du kit, Inhalt, Contenuto, Conteúdo, Contenido

The kit contains four vials of each of two levels of calibrator (0.3mL) and two vials of diluent (3mL). Each vial of calibrator contains a preparation of freeze dried human whole blood. The diluent vials contain a 0.09% solution of sodium azide.

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Storage, Conservation, Lagerung, Conservazione, Armazenamento, Almacenamiento

Stored unopened at 2 – 8 °C this product is stable until the expiration date given at the top of this page. After reconstitution, the material is stable for 1 day at 2 – 8 °C.

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Remove the stopper carefully. Using a volumetric pipette, measure 0.3mL of calibrator diluent and add to the vial to reconstitute the calibrator. Replace the stopper firmly and mix gently. Allow to stand for 5 minutes. Visually inspect that all the material has dissolved

before use. Treat the reconstituted calibrator as whole blood for the purpose of preparing samples for your glycosylated haemoglobin analysis method.

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Do not use time-expired material. Do not use incompletely dissolved material. Do not use material that has not been stored in accordance with the recommendations above.

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All the individual blood donor lots used to prepare this material were tested by FDA approved methods and found to be non-reactive for hepatitis B surface antigen (HbsAG), hepatitis C (HCV), HIV-1 and HIV-2. No test methods can offer complete assurance that these materials are free from infective agents. Handle, use and dispose of these calibrators as if they were human specimens and potentially infective.

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Assigned values

Reference System	Level 1 HbA1c Value	Level 2 HbA1c Value
National Glycohemoglobin Standardization Program "NGSP"	5.30	11.08
International Federation of Clinical Chemistry "IFCC"	3.44	9.76
Bar Coded HbA1c% Value And Lot Number	 010018700205300344	 020018700411080976